
CLARIS NON-STICK

Non-Stick Bipolarpinzetten
Non-Stick Bipolar Forceps

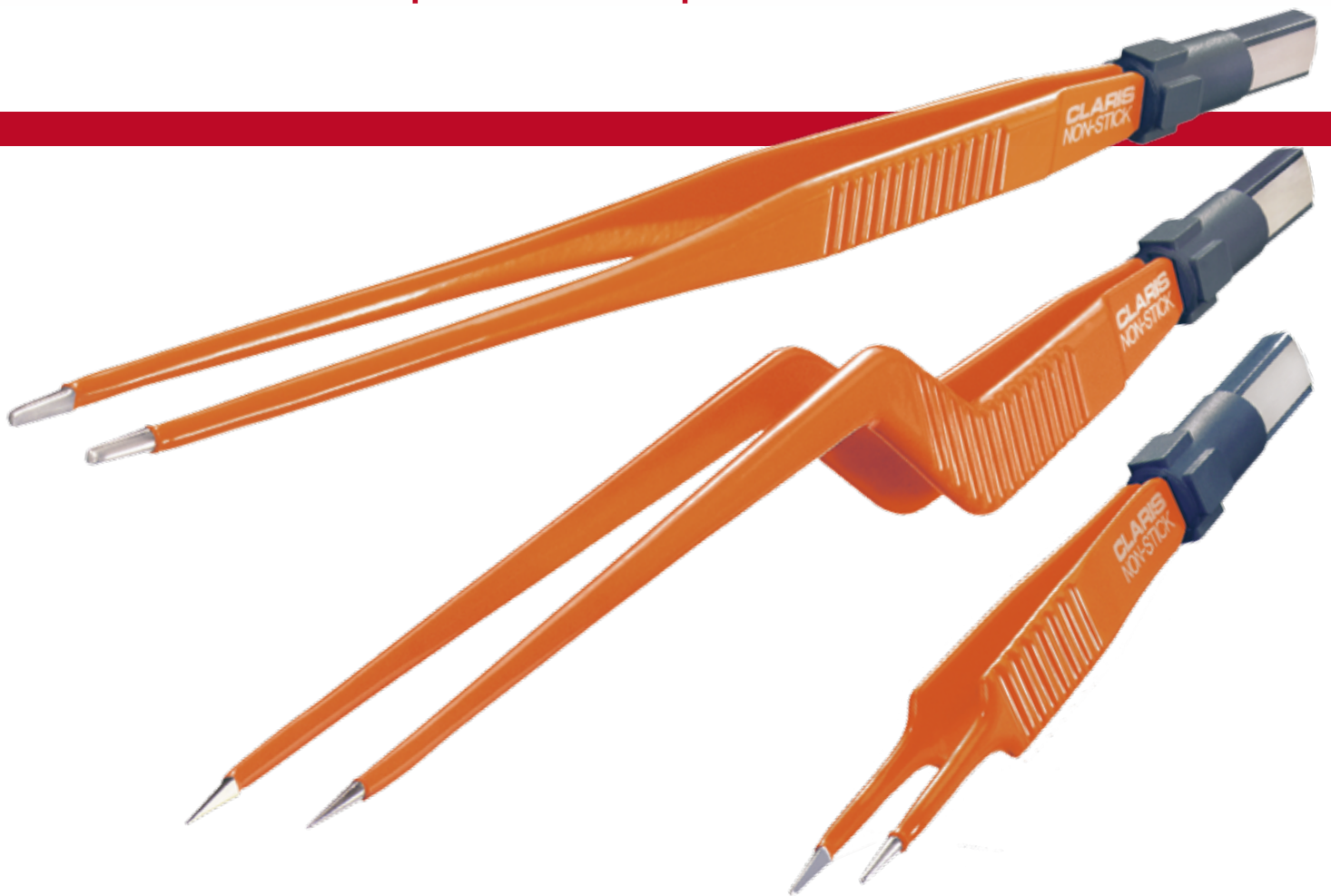CLARIS NON-STICK Bipolare Pinzetten

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CLARIS NON-STICK Bipolar Forceps

During the coagulation of tissue the coagulated tissue might stick to the tip of the forceps. This undesirable effect can almost completely be avoided by using the new CLARIS NON-STICK bipolar forceps. By applying a material with excellent thermal properties for the tips of the forceps, difficult and time-consuming cleaning of the forceps during an operation is no longer necessary and enables non-stop working. The non-stick effect is permanently ensured and will not be reduced, even if subject to frequent sterilisation. By using a suitable connecting cable, CLARIS NON-STICK bipolar forceps can be connected to all high-frequency electrosurgical generators generally used.

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The polished noble metal forceps tips are characterized by their excellent thermal conductivity which prevents the sticking of tissue. The non-stick effect is permanently ensured, as the tips are not just noble-metal coated but made of solid noble metal. Even when repolishing the surface, will the non-sticking properties be retained. High-precision shaped tips in different sizes enable pinpoint working and allow the forceps to be used for nearly all applications.

The resistant and well-proven insulation coating resists without any problems to highly alkaline cleaning agents, so that the electrical safety of the instrument is reliably ensured

A wide fluted handling surface ensures safe handling and convenient working without getting tired.

Except for the tips, the entire instrument is made of hardened stainless steel. This ensures a consistent optimum spring tension.

The plastic safety plugs enclose the complete connection area, even between the contact surfaces, in order to avoid the deposit of impurities in areas which are difficult to access.

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

Bissinger – Focus on better solutions

Bipolar and monopolar instruments
Accessories for electrosurgery
Surgical instruments
Development of instruments
to customer specifications
Service and repair

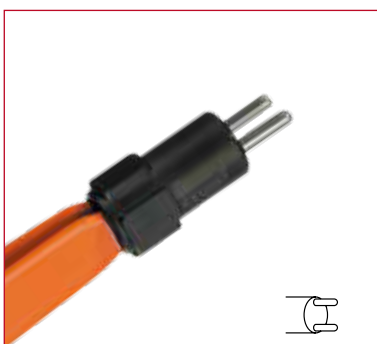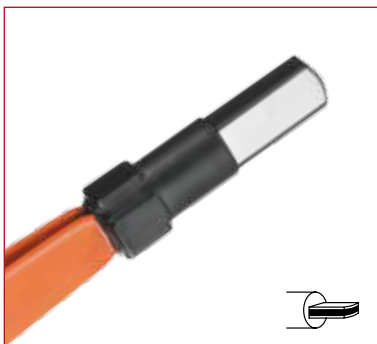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All forceps are available with
flat-plug or 2-pin-plug.

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Medizintechnik GmbH**